



Installation Checklist - For Dakim Administrators

» A Few Words

To get your Dakim units up and running as soon as possible we have included this brief list of tasks to check off as they are completed. This checklist assumes you are at least somewhat familiar with the requirements of the Dakim BrainFitness System and have begun setting up your Dakim units. Specific sections in each manual have been referenced to help you locate information quickly. Refer to the Dakim BrainFitness manual and the DakimADMIN Guide if you have any questions regarding these items.

» Before Receiving Your Dakim Units

- Confirm with your IT Professional that your network is set up so that the Dakims can communicate successfully. Refer to the Dakim BrainFitness manual (**Appendix - Network Prequalification**) for more information on specific ports.
- Use the DakimADMIN application (at www.dakimadmin.com) and follow the on-screen instructions to configure pods.
- Use the DakimADMIN application to create user profiles and assign users to pods.

» After Receiving Your Dakim Units

- Refer to the Dakim BrainFitness manual and follow the procedure outlined in section **Out of the Box**, pg. 2. for starting and connecting your Dakim to the Internet
- Use the DakimADMIN application to assign your Dakims serial numbers to pods. Refer to the DakimADMIN Manual under **Getting Started**, pg. 2, for more information.
- Connect your Dakims to the Internet during the *Data Window* between the hours of 11p.m.–7a.m. local time, and the next morning you are ready to go! for more information on the *Data Window* see the Dakim BrainFitness manual, under **Dakim Connectivity/Data Window**

» Support, Support, Support

If you require additional help, please contact Dakim via any of the following:

Customer Service: (800) 970-5067 | Support: (800) 853-7289 | support@dakim.com | www.dakim.com

